





95,00%






Specificity


Accuracy

#### The EASY CHECK from Ritter at a glance:

- For the detection of SARS-CoV-2 in saliva
- Patient-friendly due to non-invasive sample collection
- Perfect for children, the elderly and disabled people
- Rapid test results after 4 minutes (depending on room temperature)
- Producer is BfArM-listed and has a "Pre Qualification" with the WHO

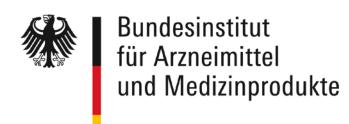



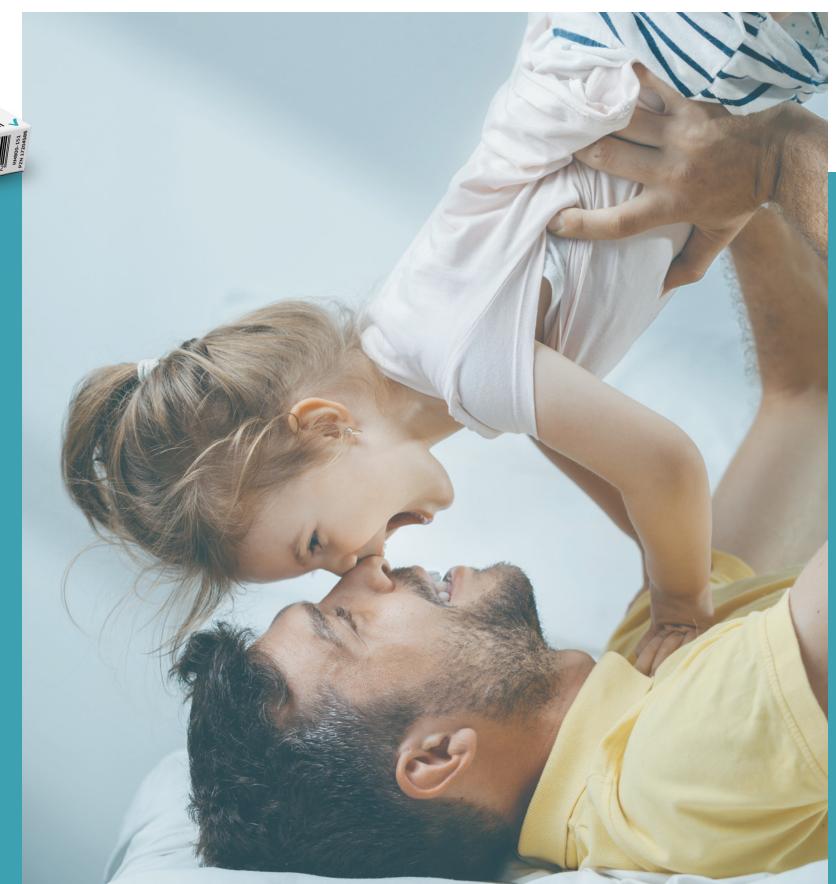





# EASY CHECK TEST KIT PACKAGE

#### One EASY CHECK pack contains:

- 20 test cassettes
- 20 pipettes
- 20 sampling tubes
- 20 disposable bags for collecting of sample material (saliva)
- Manual







#### HOW TO USE EASY CHECK

#### Instructions for use:




The EASY CHECK COVID-19 rapid antigen test can be performed with posterior oropharyngeal saliva samples.

For posterior oropharyngeal saliva samples: wash your hands with soap and water- or alcohol-based solutions.

- 1. Open the container. Clear your throat, loosen the saliva from the throat and spit it into the container (about 2ml). Avoid contamination of the outer surface of the container with saliva. The best time to collect the sample is after getting up before the patient has brushed, eaten or drunk their teeth.
- 2. Collect approximately 200µL of the saliva sample with the pipette.
- 3. Open the sample tube, add the sample and shake it to mix everything well.
- 4. Take the test device out of the pouch and place it on a table and cut off the protruding part of the sample tube lid. Then add 3 drops of the sample to the sample well and read Depending on the room temperature, you can see the result after 4-15 minutes. Fastest results at room temperature bet ween 18-30°C.





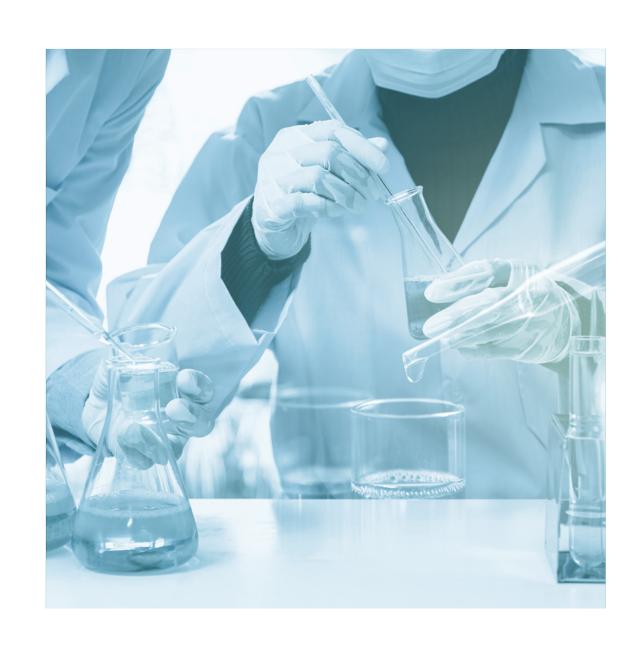



# MANUFACTURER INFORMATION

World's leading manufacturer of lateral flow test kits in China. The company is a research and development focused Chinese biotechnology company that produces high quality medical In Vitro Diagnostic (IVD) rapid test kits as well as revolutionary customerspecific reagent kits developed, manufactured and delivered to all parts of the world.

The company was founded by a team of professionals with many years of combined engineering, marketing / sales, operations and manufacturing experience in the industry.

Their in vitro diagnostic lateral flow kits screen for a wide range of targets including infectious diseases, tumors, cardiac abnormalities, substance abuse, and fertility.

Thanks to the comprehensive quality management system, which applies international standards (EN ISO 13485), a high quality of the test results and accuracy is guaranteed. Most of their products are CE and CFDA certified.





## PACKAGING















GeSino GmbH Riedbergallee 38, 60438 Frankfurt am Main Germany



#### DISTRIBUTOR

Globade GmbH
Stefan Scholz
(0049) 170 564 980 9
s.scholz@globade-trade.com
Bahnhofstraße 1, 48431 Rheine, Germany



GeSino GmbH Riedbergallee 38, 60438 Frankfurt am Main Germany



#### DISTRIBUTOR

P&P Premium Products GmbH
Michel Koren
(0049) 174 19 15 117
info@weltweite-produkte.de
Villaweg 23, 26316 Varel



GeSino GmbH Riedbergallee 38, 60438 Frankfurt am Main Germany



#### DISTRIBUTOR

MedicalAdvisors Group
Marcel Dietz
(0049) 172 65 23 841
m.dietz@medicaladvisors.group
Im Mediapark 5d, 50670 Köln



GeSino GmbH Riedbergallee 38, 60438 Frankfurt am Main Germany



#### DISTRIBUTOR

Fairhealth GmbH
Stefan Wurl
(0049) 162 51 79 79 2
stefan@fair-health.de
Gänsemarkt 43, 20354 Hamburg



GeSino GmbH Riedbergallee 38, 60438 Frankfurt am Main Germany



### DISTRIBUTOR